



## TABLE ON INDICATIONS



CATEGORY	INDICATION	ANIMAL	PRESET	DURATION	REMARK
Prophylaxis	Training preparation, warming up	Horse, Dog	2	As required	10 to 15 minutes before strain
Prophylaxis	Training, sport, competition	Horse, Dog	2	As required	10 minutes before strain
Regeneration	General	Horse, Dog	2	1 x daily	
Skin	Surface redness / inflammation	Horse, Dog	1	1 – 2 x daily	Do not apply heat !
Skin	Surface wound	Horse, Dog	1	1 – 2 x daily	Do not apply heat !
Skin	Slight swelling	Horse, Dog	9	1 x daily	Do not apply heat !
Skin	Deeper wound, dry	Horse, Dog	9	1 – 2 x daily	Do not apply heat !
Skin	Deeper wound, wet	Horse, Dog	1	1 – 2 x daily	Follow-up with preset 9
Skin	Acute wounds / injury	Horse, Dog	9	1 – 2 x daily	Do not apply heat !
Tendons / fascia	Tendon sheaths synovial fluid, windgalls	Horse, Dog	5	1 x daily	
Tendons / fascia	Slight strain	Horse, Dog	3	1 – 2 x daily	1 week, follow-up examination
Tendons / fascia	Strain / agglutination	Horse, Dog	4	2 x daily	1 week, follow-up examination
Tendons / fascia	Slight strain, cold	Horse, Dog	2	1 – 2 x daily	
Tendons / fascia	Strain, warm	Horse, Dog	9	1 – 2 x daily	When no more heat then follow-up with preset 5
Tendons / fascia	Tendon fibre rupture, initial treatment	Horse, Dog	6	1 x daily	2 weeks, follow-up with preset 5
Tendons / fascia	Tendon fibre rupture, follow-up treatment	Horse, Dog	5	1 – 2 x daily	Until healed, follow-up examination with ultrasound
Tendons / fascia	Suspensory ligament, warm	Horse	9	1 – 2 x daily	When no more heat then follow-up with preset 6
Tendons / fascia	Suspensory ligament, cold	Horse	6	1 – 2 x daily	
Tendons / fascia	Strain / irritation of the biceps tendon	Horse, Dog	4	2 x daily	
Musculature	Preventive	Horse, Dog	2	As required	10 minutes before strain
Musculature	Warming up	Horse, Dog	2	As required	10 to 15 minutes before strain
Musculature	Promotion of blood circulation	Horse, Dog	8	As required	10 minutes before strain
Musculature	Muscular tension, general	Horse, Dog	2	1 x daily	1 week, follow-up examination, change to preset 7 *
Musculature	Muscular tension, cervical spine	Horse, Dog	2	1 – 2 x daily	1 week, follow-up examination, change to preset 7 *
Musculature	Muscular tension, lumbar spine	Horse, Dog	2	1 – 2 x daily	1 week, follow-up examination, change to preset 7 *
Musculature	Muscular tension, neck – shoulder	Horse, Dog	2	1 – 2 x daily	1 week, follow-up examination, change to preset 7 *
Musculature	Muscular tension, shoulder – arm	Horse, Dog	2	1 – 2 x daily	1 week, follow-up examination, change to preset 7 *
Musculature	Muscular tension, gluteal muscles	Horse, Dog	2	1 – 2 x daily	1 week, follow-up examination, change to preset 7 *
Musculature	Muscular tension, long ischium	Horse, Dog	2	1 – 2 x daily	1 week, follow-up examination, change to preset 7 *
Musculature	Muscular tension, quadriceps	Horse, Dog	2	1 – 2 x daily	1 week, follow-up examination, change to preset 7 *
Musculature	Slight strain	Horse, Dog	3	2 x daily	1 week, follow-up examination, change to preset 7 *
Musculature	Strain	Horse, Dog	4	2 x daily	1 week, follow-up examination, change to preset 7 *
Musculature	Muscle rupture, initial treatment	Horse, Dog	6	2 x daily	2 weeks
Musculature	Muscle rupture, follow-up treatment	Horse, Dog	5	2 x daily	2 – 4 weeks, change to preset 4 or 3
Nerves	Crushing injury, regeneration	Horse, Dog	6	1 x daily	4 – 6 weeks
Joints	General, small amount of synovial fluid	Horse, Dog	2	1 x daily	
Joints	General, average amount of synovial fluid	Horse, Dog	3	1 x daily	2 weeks, then change to preset 2
Joints	General, avg. amount of synovial fluid, pain	Horse, Dog	4	1 x daily	
Joints	General, large amount of synovial fluid, pain	Horse, Dog	5	1 – 2 x daily	2 weeks, follow-up examination
Joints	Spondylosis cervical spine	Dog	2	1 – 2 x daily	Change to preset 6 *
Joints	Spondylosis lumbar spine	Dog	2	1 – 2 x daily	Change to preset 6 *
Joints	Pain / Elbow arthrosis	Horse, Dog	2	1 x daily	
Joints	Instability in the sacroiliac joint	Horse, Dog	2	1 – 2 x daily	
Joints	Pain / arthrosis in the hip joint	Horse, Dog	2	1 – 2 x daily	
Joints	Pain / karpal joint arthrosis	Horse	2	1 – 2 x daily	
Joints	Pain / knee joint arthrosis	Horse, Dog	2	1 – 2 x daily	
Joints	Pain / shoulder joint arthrosis	Horse, Dog	2	1 – 2 x daily	
Joints	Pain / ankle joint arthrosis	Dog	2	1 – 2 x daily	
Joints	Spavin	Horse	7	1 x daily	If preset 7 is too strong, change to preset 2
Joints	Distorsions	Horse, Dog	4	1 – 2 x daily	1 week, follow-up examination
Joints	Dysplasia of the hip	Dog	2	1 x daily	
Pain	i.e. FLUTD	Cat	2	1 – 2 x daily	Treatment of the initial disease necessary !
Pain	Kissing Spine Syndrome	Horse	2	2 x daily	2 weeks, follow-up examination
Pain	Akuter pain or cramps	Horse, Dog	8	immediately	
Acupuncture	Activation of acupuncture and trigger points	Horse, Dog	7	2 x weekly	

\*) if applicable